


回拉式安全自毁注射器 Needle retractable safety syringe

+ 产品优势 / Product advantages

- 45° 角安全回弹锁卡式自动隐藏针头
- 防止血液接触/防止医源性污染
- 防止针刺伤/防止锐器废弃针刺伤
- 单手操作，单手激活
- 无需改变注射操作习惯
- 更有竞争力优势的价格



- Fingers stay behind the needle at all times
- Non-exposure of needle after manual retraction
- Requires minimum training
- No add-on pieces allows for low angle injections
- Retract speed under control - no splatter after activation
- Competitive price advantage over other safety technologies

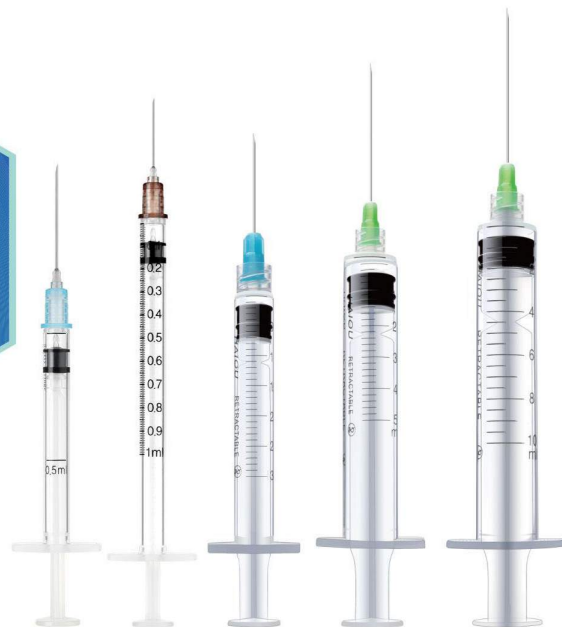

Also available with other needle size on request
针头也可根据要求更换大小

一次性使用回拉式自毁注射器 / Specification

0.05-0.5ml	1ml	2-3ml	5ml	10ml
------------	-----	-------	-----	------

一次性使用无菌自毁型固定剂量疫苗注射器 / Specification

0.05ml	0.1ml	0.2ml	0.3ml	0.4ml	0.5ml	1ml
--------	-------	-------	-------	-------	-------	-----

胰岛素U-100或U-40 / Insulin Syringe size (U-100 or U-40)

0.3ml	0.5ml	1ml
-------	-------	-----

针头规格 / Needle gauge

18G	19G	20G	21G	22G	23G
24G	25G	26G	27G	29G	30G

+ 操作参考图 / How to use

<p>抽药前旋紧针座，使用标准程序抽取药液</p> <p>Be sure needle fix tightly and draw up medicine using standard procedure</p>  <p>1</p>	<p>注射完成的同时针管被芯杆锁住</p> <p>Give injection using aseptic techniques, fully depress the plunger to lock the needle</p>  <p>2</p>
<p>将芯杆回拉，使针头完全缩回外套</p> <p>Pull plunger back until needle fully be retracted into the barrel.</p>  <p>3</p>	<p>沿切口处折断芯杆，针头得到防护。</p> <p>Snap off the plunger, the used needle will be encased inside the barrel</p>  <p>4</p>